

# Cleaning and Disinfection Guide for MRI System

- Use disposable protective gloves to prevent infection
- Use a soft cloth or gauze to wipe the system
- After disinfection, allow the wiped surface to dry
- Ventilate the scan room after disinfection
- Do not spray disinfectant directly onto the system, coils or other accessories
- Use the recommended disinfectant



Recommended disinfectant

Disinfectant	Concentration
Ethyl Alcohol	70 %



Clean all buttons and surrounding areas.  
Please avoid inadvertently pressing the buttons.



Clean the gantry and inside of the bore.  
Do not use alcohol on acrylic parts.  
Acrylic is used for the projector cover inside the gantry and phantoms.




Always remove RF coil from connector port before cleaning.



Thoroughly clean and wipe the coil surface.  
Do not use alcohol to wipe sections to which adhesive is applied (label, Velcro fastener, etc.).



Thoroughly clean and wipe the patient table and mat.  
Do not clean or disinfect the connector port.

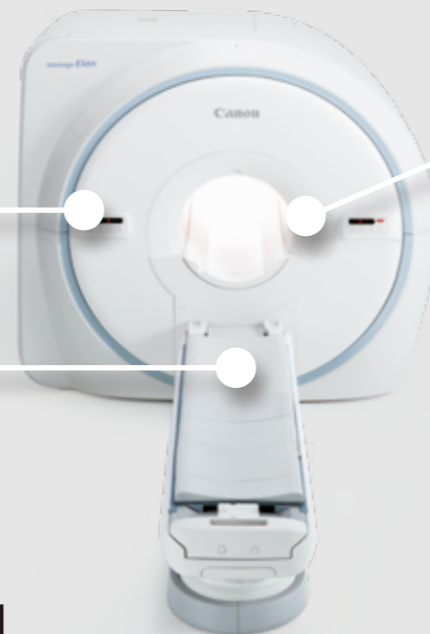

Thoroughly clean and wipe the keyboard, mouse and control pad. Please avoid inadvertently pressing the Emergency button.

Please refer to the operation manual for the applicable system for further details.